

ACTOSAL® FLÄCHE AF

Floor and Surface Disinfectant

- ✓ Contains QAC
- ✓ Aldehyde and phenol free
- √ Suitable for use in any type of surface
- ✓ Bactericidal, fungicidal and virucidal action
- ✓ A concentrated product



Actosal® Fläche AF is a concentrated product that cleans and disinfects simultaneously with chemical content. Free of aldehyde and phenol. Suitable for use on all surfaces.









ACTOSAL® FLÄCHE AF

Floor and Surface Disinfectant



Areas of Application:

In risky areas of hospitals: it is used for cleaning and disinfection of all water-resistant floors, surface and fixtures, hard and PVC floors, acrylic glasses, incubators, slides, urinary containers and kitchens contaminated with blood and body secretion.

Method of Use:

It is applied by mopping for disinfection and cleaning processes of floors, and by wiping for other surfaces. It should be applied after contamination.

Always complete the disinfection and cleaning process by wiping from clean areas to dirty areas, from farther areas to closer areas, and from top to bottom. Make sure that the surface is completely wet.


Chemical Composition:

100 g contains: 9.9 g of diethyldimethylammonium chloride (70 %) and auxiliary substances.


Instructions For Use:




Wear suitable protective gloves, goggles and clothing.

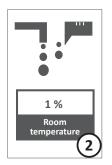

Add the product to the water at the recommended concentration

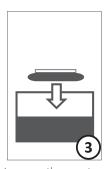

Immerse the mop to the solution.

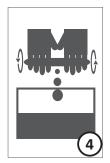

Squeeze the mop.



Start the disinfection process.



Wait for the action time, allow to dry.

Concentration of Use				
Concentration Ratio	Water Amount	Product Amount		
1 %	990 ml	10 ml		
Packaging				
Unit	Pieces in a Box	Article-No.		
1 L	14	05.5502.1		
5 L	4	05.5502.5		

Microbiological Activity	Test Method	Concentration	Effect Time
Bactericidal	EN 1276	1 %	15 min.
Fungicidal (<i>C. albicans, A. niger</i>)	EN 1650	1 %	15 min.
Virucidal (Vaccinia)	EN 14476	1 %	15 min.



ACTO GmbH
Head Office & Factory
Schulstrasse 8 • 31613 Wietzen-Holte / Germany
Tel: +49 (0) 5022 9801-0 • Fax: +49 (0) 5022 9801-37

Braunschweig Office

Büchnerstrasse 11 • 38118 Braunschweig / Germany

