




Wear suitable protective gloves, goggles and clothing.

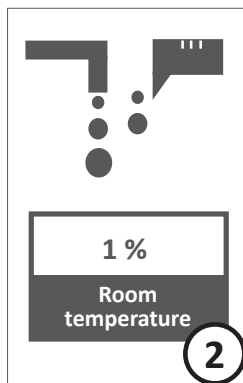


Add the product to the water at the recommended concentration.

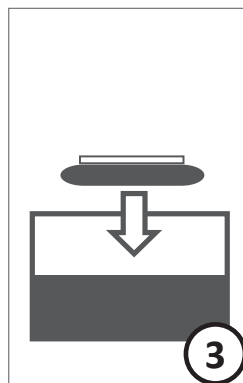


Immerse the mop to the solution.

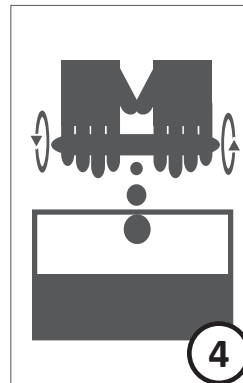

Squeeze the mop.

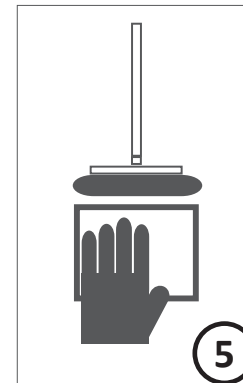

Start the disinfection process.

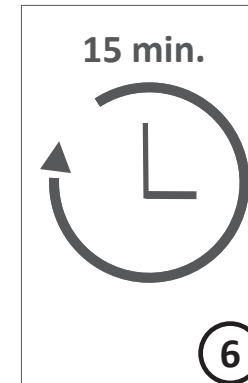

Wait for the action time, allow to dry.

Actosal® Fläche AF is a concentrated product that cleans and disinfects simultaneously with chemical content. Free of aldehyde and phenol. Suitable for use on all surfaces.

Areas of Application:

In risky areas of hospitals: it is used for cleaning and disinfection of all water-resistant floors, surface and fixtures, hard and PVC floors, acrylic glasses, incubators, slides, urinary containers and kitchens contaminated with blood and body secretion.

Method of Use:

It is applied by mopping for disinfection and cleaning processes of floors, and by wiping for other surfaces. It should be applied after contamination. Always complete the disinfection and cleaning process by wiping from

clean areas to dirty areas, from farther areas to closer areas, and from top to bottom. Make sure that the surface is completely wet.

Chemical Composition:

100 g contains: 9.9 g of diethyldimethylammonium chloride (70 %) and auxiliary substances.

Microbiological Activity	Test Method	Concentration	Effect Time
Bactericidal	EN 1276	1 %	15 min.
Fungicidal (<i>C. albicans</i> , <i>A. niger</i>)	EN 1650	1 %	15 min.
Virucidal (Vaccinia)	EN 14476	1 %	15 min.

Concentration of Use		
Concentration Ratio	Water Amount	Product Amount
1 %	990 ml	10 ml

▼
For warnings and precautions, see the label and user manual.