



» ALCOHOL BASED RAPID SURFACE DISINFECTANTS «

ACTOSEPT® AF CITRUS

Rapid Surface Disinfectant for Medical Devices

- ✓ Containd alcohol (30 %) and QAC
- ✓ Bactericidal, fungicidal and virucidal
- ✓ Aldehyde and phenol free
- ✓ Easy to use
- ✓ High material compatibility
- ✓ Fast drying feature



Actosept® AF Citrus is suitable for cleaning and disinfection of medical devices.



German Company



ACTO® Pharma
Cosmetic
Disinfection
Wound Care
Food Supplement
Active Chemical Technology Organisation

ACTOSEPT® AF CITRUS

Rapid Surface Disinfectant for Medical Devices



Areas of Application:

Can be used safely on any alcohol-resistant medical device surface. Also suitable for the disinfection and cleaning of medical device surfaces used in dental field.

Method of Use:


Spray the surface to be disinfected from a distance of 30 cm and ensure that the surface is completely covered. Wait 1-5 min. for it to take effect. Always complete the disinfection and cleaning process by wiping from clean areas to dirty areas, from farther areas to closer areas, and from top to bottom with a clean and dry cloth. Make sure that the surface is completely wet.

Chemical Composition:


100 g contains:

20 g Ethyl alcohol, 10 g 2-Propanol,
0.05 g didecyldimethylammonium propionate,
nonionic surfactants and auxiliary substances.


Instructions For Use:

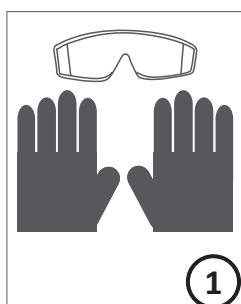


Wear suitable protective gloves, goggles and clothing.

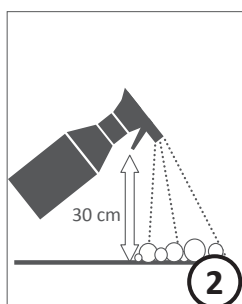

Spray the surfaces to be disinfected from a distance of 30 cm and ensure that the surface is completely covered.

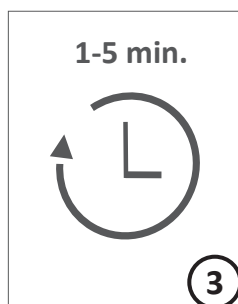

Wait for the recommended action time.

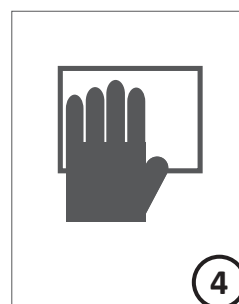

Always complete the disinfection and cleaning process by wiping from clean areas to dirty areas, from farther areas to closer areas, and from top to bottom with a clean and dry cloth. Make sure that the surface is completely wet.

Packaging		
Unit	Pieces in a Box	Article-No.
500 ml	18	05.2401.500
1 L	12	05.2401.1
4500+500 ml	3	05.2401.45
5 L	4	05.2401.5

Microbiological Activity	Test Method	Concentration	Effect Time
Bactericidal	EN 13727	Ready-to-use	1 min.
Yeasticidal (<i>C. albicans</i>)	EN 13624	Ready-to-use	1 min.
Fungicidal (<i>A. niger</i>)	EN 13624	Ready-to-use	5 min.
Limited Virucidal (<i>Vaccinia</i>)	EN 14476	Ready-to-use	5 min.

