




Wear suitable protective gloves, goggles and clothing.

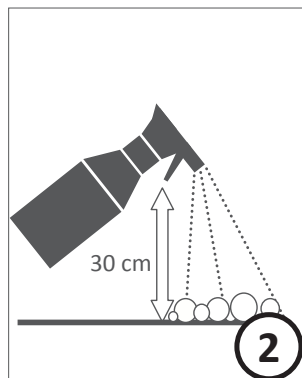

Spray the surfaces to be disinfected from a distance of 30 cm and ensure that the surface is completely covered.

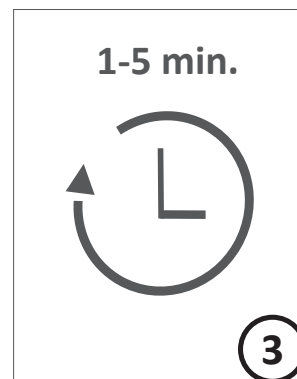

Wait for the recommended action time.

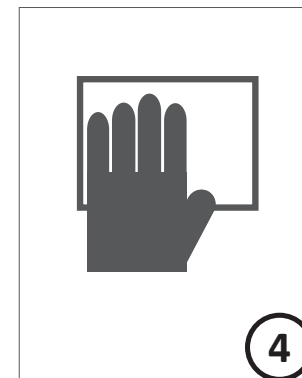

Always complete the disinfection and cleaning process by wiping from clean areas to dirty areas, from farther areas to closer areas, and from top to bottom with a clean and dry cloth. Make sure that the surface is completely wet.

Actosept® AF Citrus is suitable for cleaning and disinfection of medical devices.

### Areas of Application:

Can be used safely on any alcohol-resistant medical device surface. Also suitable for the disinfection and cleaning of medical device surfaces used in dental field.

### Method of Use:

Spray the surface to be disinfected from a distance of 30 cm and ensure that the surface is completely covered. Wait 1-5 min. for it to take effect. Always complete the disinfection and cleaning process by wiping from clean areas to dirty areas, from farther areas to closer areas, and from top to bottom with a clean and dry cloth. Make sure that the surface is completely wet.

### Chemical Composition:

100 g contains:

20 g Ethyl alcohol, 10 g 2-Propanol,

0.05 g didecyltrimethylammonium propionate, nonionic surfactants and auxiliary substances.

Microbiological Activity	Test Method	Concentration	Effect Time
Bactericidal	EN 13727	Ready-to-use	1 min.
Yeasticidal ( <i>C. albicans</i> )	EN 13624	Ready-to-use	1 min.
Fungicidal ( <i>A. niger</i> )	EN 13624	Ready-to-use	5 min.
Limited Virucidal ( <i>Vaccinia</i> )	EN 14476	Ready-to-use	5 min.

Packaging		
Unit	Pieces in a Box	Article-No.
500 ml	18	05.2401.500
1 L	12	05.2401.1
4500+500 ml	3	05.2401.45
5 L	4	05.2401.5

▼  
For  
warnings  
and  
precautions,  
see the  
label  
and user  
manual.