




Wear suitable protective gloves, goggles and clothing.

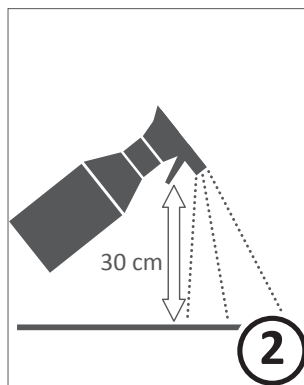

Spray the surfaces to be disinfected from a distance of 30 cm and ensure that the surface is completely covered.

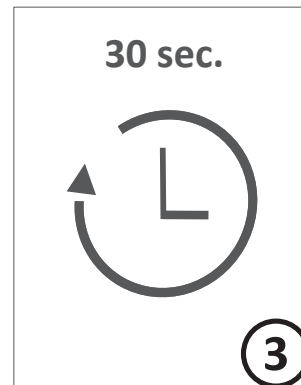

Wait 30 seconds for it to take effect

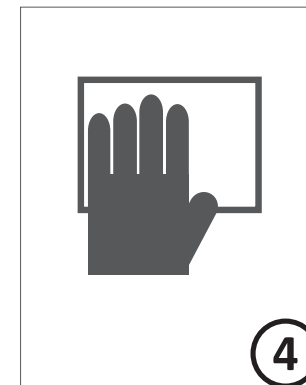

Always complete the disinfection and cleaning process by wiping from clean areas to dirty areas, from farther areas to closer areas, and from top to bottom with a clean and dry cloth. Make sure that the surface is completely wet.

**Actosept® AF** is a fast acting ready-to-use surface disinfectant which contains alcohol. Ethanol, 2-propanol and didecyldimethylammonium propionate containing product can be used for the fast disinfection of small surfaces with its widely effective spectrum and easy use.

### Areas of Application:

It is used for the fast disinfection of all kinds of surfaces except acrylic glass and alcohol-sensitive surfaces. It is suitable for the nursing desks, tables and chairs, medicine trolleys, areas where hands and skin can frequently contact, and the patient care units. It smells fresh and nice.

### Method of Use:

Spray the surface to be disinfected from a distance of 30 cm and ensure that the surface is completely covered. Wait 30 seconds for it to take effect. Always complete the disinfection and cleaning process by wiping from clean areas to dirty

areas, from farther areas to closer areas, and from top to bottom with a clean and dry cloth. Make sure that the surface is completely wet.

### Chemical Composition:

100 g contains:

50 g ethanol, 10 g 2-propanol,

0.05 g didecyldimethylammonium propionate, nonionic surfactants and auxiliary substances.

Microbiological Activity	Test Method	Concentration	Effect Time
Bactericidal (Inc. MRSA, VRE, A. baumannii)	DGHM, EN 13727, EN 16615	Ready-to-use	30 sec.
Yeasticidal (C. albicans)	DGHM EN 13624, EN 16615	Ready-to-use	30 sec.
Tuberculocidal, Mycobactericidal	EN 14348	Ready-to-use	30 sec.
Limited Virucidal (Corona, Rota)	EN 14476	Ready-to-use	30 sec.

▼  
For  
warnings  
and  
precautions,  
see the  
label  
and user  
manual.