



» ALCOHOL BASED RAPID SURFACE DISINFECTANTS «

ACTOSEPT® AF TÜCHER

Alcohol-Based Ready-to-Use Wipes for Rapid Surface Disinfection

- ✓ Alcohol-based wipes
- ✓ Disinfects fast and dries rapidly
- ✓ Effective in 30 sec. against bacteria, fungi, mycobacteria and enveloped viruses
- ✓ Ready-to-use
- ✓ High material compatibility
- ✓ Aldehyde and phenol free
- ✓ The impregnation solution **Actosept® AF** is **DGHM/VAH** listed
- ✓ The impregnation solution **Actosept® AF** is dermatologically tested and found to be "Excellent" in the German **DERMATEST** laboratory



Actosept® AF Tücher is used for disinfection of the surfaces of alcohol resistant medical devices which are not suitable for contact with liquid. It is suitable for cleaning large areas, in the form of wipes containing nonwovens (viscose/polyester). It does not contain aldehyde and phenol.



German Company



ACTO® Pharma
Cosmetic
Disinfection
Wound Care
Food Supplement
Active Chemical Technology Organisation

ACTOSEPT® AF TÜCHER



Alcohol-Based Ready-to-Use Wipes for Rapid Surface Disinfection

Areas of Application:

It is used for disinfection of alcohol resistant medical devices with a non-invasive, regular and smooth surface. Suitable for safe and effective disinfection of medical devices such as dental units (excluding drills and hoses), anesthesia devices, operating room lamps and desks that need to be disinfected frequently.

Method of Use:


Wipe the pre-cleaned surface of the medical device with **Actosept® AF Tücher** to use 3-4 wipes per square meter. Make sure that the wipes reach the entire surface of the device and there is no dry place left. Wait for the duration of the effect. Do not dry after use.

The packaging should be tightly closed after each use. Used wipes should not be reused.

Chemical Composition:

Content of 100 g Impregnation Solution:
50 g Ethanol, 10 g 2-Propanol, 0.05 g
Didecyldimethylammonium propionate and
excipients.


Instructions For Use:

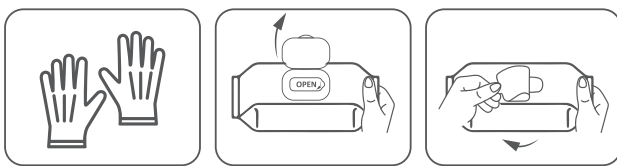

Wear gloves

Open the flap lid

Open the foil

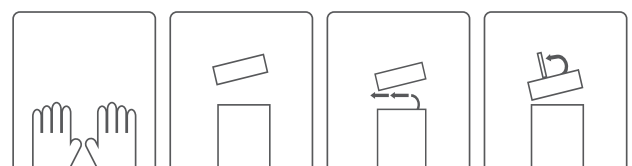


Wear gloves

Remove the
cap

Remove the
foil

Open the
hinged lid

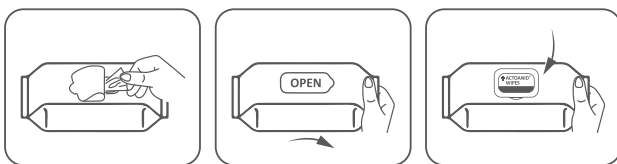

Take a wipe

Close the foil

Close the lid tightly



Put the towel
through the
hinged lid

Fasten the cap

Tear off the
towel

Fasten the
hinged lid

Packaging		
Unit	Pieces in a Box	Article-No.
100 Wipes - 15x24 cm	6 (PE Container)	05.3101.100
80 Wipes - 15x20 cm	18 (Packaging Bag)	05.3101.80

Microbiological Activity	Test Method	Effect Time
Bactericidal	EN 13727* EN 16615	30 sec.
Fungicidal (<i>Aspergillus brasiliensis</i> , <i>Candida albicans</i>)	EN 16615	30 sec.
Tuberculocidal, Mycobactericidal	EN 14348*	30 sec.
Limited Virucidal (<i>Corona</i> , <i>Rota</i>)	EN 14476*	30 sec.

* Tests were made with impregnation solution.

